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HinriPoly Polyurethane 1:1

Instructions for use

Product description: A two component (1:1) model material based on polyurethane with a low shrinkage for the production of demonstration and presentation models and for internal control models as well.

- very low-viscous (thin flowing)
- easy to process
- long working time
- low shrinkage < 0,1 %
- stable in dimension
- high precision and edge stability
- easy to mill and grind
- optionally dyeable in different colours

Technical data:

| | |
|-----------------------|--|
| Mixing ratio A : B | 1 : 1 |
| Viskosity | 65 mPas |
| Working time at 23 °C | 3 – 4 minutes |
| Demouldable at 23 °C | > 30 minutes |
| Compressive strength | 80 MPa |
| Shrinkage | < 0.1 % |
| Density | 1.05 g/cm ³ |
| Colour | beige |
| Storage | 12 months in original sealed container |

Application: Before using the model material please mix the same thoroughly until it has got a homogeneous appearance. Mix both components thoroughly with each other at room temperature (> 18 °C) according to the indicated 1.1 mixing ratio. Before casting please make sure that the parts and moulds are free from humidity and non-evaporated release agent.

Important: To prevent unwanted reactions, such as premature curing, never measure out the two components using the same measuring cylinder or confound the lids. Close the container again carefully after each use.

Dangerous reactions: No known dangerous reactions, if stored and used properly.

Toxicity: None known

Storage: Store containers tightly sealed in a well-ventilated place. Keep containers dry. Keep away from water, alkalis, strong acids and oxidising agents. The material should be kept in tightly closed original receptacles at temperatures of 15 - 25 °C.

Ecological information: Present experience suggests that there is no detrimental effect on the environment, if it is used properly and current regulations are observed. It is non-biodegradable and insoluble in water when cured. It can be easily separated from water by filtering.

Colouring:

For Colouring of Polyurethan we can offer you following colour paste :

| | | | |
|--|------------|------------------------------|------------|
| white, 100 ml dosage bottle | REF 243004 | black, 100 ml dosage bottle | REF 243005 |
| red, 100 100 ml dosage bottle | REF 243006 | yellow, 100 ml dosage bottle | REF 243007 |
| blue, 100 100 ml dosage bottle | REF 243008 | green, 100 ml dosage bottle | REF 243009 |
| Colour Set, 6 x 100 ml 100 ml dosage bottles | | | REF 243011 |

The maximum colour content should not reach more that 5 %.

Further questions: Contact our technical services (Tel.: +49 (0) 53 21 – 5 06 24 / 25) or our sales representatives.

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