## Hinrtray

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## Hinritray

## Working instructions

Hinritray is a light curing tray material in form of prefabricated, upper baseplates to make rigid, custom impression trays and bite-blocks, as well as for other indications (refer to "Indications"). Hinritray can be cured in all ultraviolet and Halogen light curing units. The baseplates are supplied in a re-usable light-proof box which can be used for transporting or storing models etc.

**Packing size:** 1 box contains 50 blanks

Hinritray, pink - REF 801000    Hinritray, transparent (mint) - REF 801020    Hinritray, blue - REF 801010

### Indications:

- Custom impression tray
- Baseplate for setting up dentures
- Bite wafer for trying in framework
- Bite-block
- Articulator guide pin excursion tracing
- Splinting outer coping or telescoping crowns

### Preparation:

- Mark the outline of the future tray on the model.
- Block out the undercuts on the model.
- To create a spacer for the impression material, block out the entire model (e.g. with a sheet of wax) and cut stops out.
- If the gypsum is very dry, soak the model briefly with water or apply dye isolation or an alginate isolation.

### Application:

- Remove the baseplate from the box and close it.
- Once the model has been blocked out, adapt the baseplate to it and cut to shape using, e.g. a scalpel.
- Use the remaining material to form a handle and attach it to the tray.
- Place the model and tray in a light curing unit and cure without vacuum.
- Remove the tray from the model and polymerize its underside.
- If the tray has been cured in air, use alcohol to remove the smear layer.

### Times:

Working at daylight 20 minutes. Polymerisation in an ultraviolet or Halogen light curing unit, depending on the unit in use:

1. The top of the tray, on the model 3-5 minutes
2. The underside of the tray, off the model 3-5 minutes

### Trimming:

- Always use a dust extractor when trimming the tray!
- Trim the custom tray with tungsten carbide burs and smooth it with sandpaper. Pumice slurry can be used for polishing.

**Caution:** Avoid any contact between the eyes and skin and the uncured material, i.e. also with the inhibited layer. The product has been developed for use in the dental field and must be used in accordance with the usage directions. The manufacturer accepts no liability for damage that results from use other than the above. Furthermore the user is obliged under his own responsibility to examine the material for its suitability and its usage possibilities for the intended applications, especially if the latter are not detailed in the usage directions.

**Hygiene:** Immerse impressions taken with Hinritray custom trays in a disinfectant solution based on glutaraldehyde (the time depends on the disinfectant manufacturer's directions).

**Incompatibility:** Hinritray may cause sensitization in sensitive persons. Against the containings of Hinritray (Hinritray contains acrylates). If such reactions are experienced, discontinue the use of Hinritray.

**Technical Data:** Flexural strength min. 130 N/mm<sup>2</sup>

**Storage:** Do not store Hinritray above 22°C. Protect against light.

**Shelf-life:** Do not use after the expiry date.

Date of information: 10/2015